Kartell

INSTRUCTIONS FOR USE

SAMPLE CONTAINERS:

TRADE NAME	Cylindrical and conical tubes in PP, graduated conical tubes in PP and graduated conical tubes in PS
IDENTIFICATION	REUSABLE AUTOCLAVABLE TUBES FOR CENTRIFUGES
INTENDED USE	Devices for professional laboratory analysis use that function as the ideal recipients to contain blood and other biological fluids destined for laboratory analysis and use in chemistry for the measurement of the chemical, physical and microscopic characteristics of the samples drawn. Ideal for use in centrifuges for analyses where separation of components into different densities is required.
NON AUTOMATED DEVICE	----
QUANTITATIVE DEVICE	The results of analyses conducted on the samples contained in the device must always be subject to the scrutiny of highly qualified personnel for validation and report.
SAMPLE	Biological sample or sample of another nature
IN VITRO MEDICAL DIAGNOSTIC DEVICE	Compliant with (EU) regulation 2017/746 for in vitro medical-diagnostic devices
USERS	Highly qualified personnel. For professional laboratory analysis use
STORAGE	Protect from direct sunlight and UV rays. Store in a dry location at room temperature
MATERIAL STABILITY	The plastic materials used allow the product to remain stable over time. It is established that these types of devices are valid for 5 years from the production date listed on the label.
USAGE INSTRUCTIONS AND RESTRICTIONS	<p>comply with laboratory and/or environmental protocols in which the item is used for the execution of the analyses.</p> <p>Do not fill the tubes up to the border to avoid leakage, especially in cases of infective biological material</p> <p>To ensure safe use, use the tubes together with their specific support.</p> <p>Verify the speed limit that the tubes may be subjected to in the centrifuge by referring to the indications provided in the catalogue and in the technical data sheets.</p> <p>Seal caps well before placing in centrifuge</p> <p>Report any defects that prevent use of the tubes to the product distributor</p> <p>Report serious accidents to the proper authorities.</p> <p>Tubes are reusable after procedure in autoclavable at 121°C for 20 minutes</p> <p>Do not approach open flames or heat sources</p> <p>Dispose of tubes according to the applicable law.</p> <p>Wear gloves and personal protective equipment when handling potentially contaminating solutions.</p> <p>Keep out of children's reach</p>
REUSABLE DEVICES	Tubes are reusable after procedure in autoclavable at 121°C for 20 minutes

Kartell SpA
 Via delle Industrie 1
 20082 Noviglio (MI)
 Tel. 02/900121
 Website: www.kartelllabware.com

to access instructions in other languages:
www.kartelllabware.com/en/download/

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